



Instructions for use HeltiQ Scar Gel

- Carefully read the instructions before using this product - Do not use on children under 2 years of age - For external use only -

CONTENTS INSTRUCTIONS FOR USE

- | | |
|-----------------------------|--|
| 1. Contents HeltiQ Scar Gel | 6. Possible side effects |
| 2. HeltiQ Scar Gel | 7. Contra-indications |
| 3. Application | 8. Precautions |
| 4. Ingredients | 9. Complaints and customer service support |
| 5. Instructions for use | |

1. CONTENTS HELTIQ SCAR GEL

1 Flacon 1 oz / 30 ml

1 Instructions for use

2. HELTIQ SCAR GEL

HeltiQ Scar Gel is a medical device and helps with the treatment of scar tissue resulting from small and large(r) injuries. The scar gel also relieves the itching, pain and redness of scars. HeltiQ Scar Gel has been proven to be effective for the treatment of hypertrophic and keloidal scars, dries quickly and maintains hydration of the skin. Only apply the scar gel on completely closed scars. For optimum results use the scar gel 2x per day for a minimum period of 2 months. For external use only.

3. APPLICATION

HeltiQ Scar Gel is used for the treatment and prevention of keloids and hypertrophic scars (like scars that occurred as a result of injuries, general surgical procedures and burns) after the wound has healed and the skin is closed again.

4. INGREDIENTS

Aqua, Dimethicone, Prunus Amygdalus Dulcis Oil, Cetearyl Alcohol, Echium Plantagineum Seed Oil, Propanediol, Pentylene Glycol, Limnanthes Alba Seed Oil, Glycerin, Cetearyl Oliviate, Sorbitan Oliviate, Potassium Palmitoyl Hydrolyzed Wheat Protein, Glyceryl Stearate, Pullulan, Helianthus Annuus Seed Oil, Lactic Acid, Pectin, Sorbitan Caprylate, Tocopherol.

HeltiQ Scar Gel contains various silicones that have been classified as "medical grade". This means that these silicones are suitable for use on human skin.

5. INSTRUCTIONS FOR USE

Clean and dry the area that needs to be treated. HeltiQ Scar Gel may only be applied to clean, dry and closed skin. For an effective result apply the gel both in the morning and in the evening and remove the excess gel. Have the scar gel dry well to prevent staining of clothes. After a brief time of drying it is possible to apply cosmetics on the treated area. Only apply the scar tissue gel on completely closed scars. For optimal results use the HeltiQ Scar Gel a minimum of 2 months.



Instructions for use HeltiQ Scar Gel

- Carefully read the instructions before using this product - Do not use on children under 2 years of age - For external use only -

REF 138316
552264 - V1.20160621

6. POSSIBLE SIDE EFFECTS

In exceptional cases the gel may cause redness, pain or irritation of the skin. If you are concerned about this or other side effects, please contact your pharmacist or doctor. You can report side effects via: info@utermohlen.nl. Please contact your doctor if an inflammation and/or other complication occurs.

7. CONTRA-INDICATIONS

HeltiQ Scar Gel may not be used in the following situations:

- On open, unhealed wounds
- In the area of or in contact with the mucous membranes
- In the area of or too close to the eyes
- For other skin disorders without first consulting your doctor

8. PRECAUTIONS

- For use on children over 2 years of age the scar tissue gel should only be applied by adults
- Do not have children apply the scar tissue gel themselves
- Keep out of the reach and sight of children
- Store in a cool and dry place (below 25°C, not close to heat sources)
- Not to be ingested
- Not for internal use
- Do not use 3 years after the date mentioned on the box and the flacon

9. COMPLAINTS AND CUSTOMER SERVICE SUPPORT

Contact the manufacturer or supplier with questions or to report an adverse reaction. Complaints and questions can also be reported via www.heltiq.com/scargel.



Since 1880

Manufactured by:

 Koninklijke Utermöhlen NV
De Overweg 1, 8471 ZA Wolvega,
The Netherlands, www.heltiq.com

